GB ENZYMAL LIQUID (IMS-1222, IMS-1224, IMS-1226)**Application:** Enzymatic Cleaner liquid**Net Contents:** IMS-1224: Quart Bottle, 950 ml with pump
IMS-1226: 3,8 liters bottle
IMS-1222: Box of 40 Liquid Packets of 10 ml (concentrate)

Instructions for use: Shake well before using. Wear gloves and eye protection. Use with water 32°-60°C, demineralized water is ideal. **Pre-soak:** Add 30ml of concentrate per 3.8L of water. Bottles with pump insert: pump one time per liter of water. Single packet: 1pck (10ml) per 3,5 liter of water. Soak for 3-5 minutes. Rinse, dry thoroughly and proceed with sterilization. **Ultrasonic Cleaning:** Add 30ml of concentrate per 3.8L of water. Bottles with pump insert: pump one time per liter of water. Single packet: 1pck (10ml) per 3,5 liter of water. After cleaning, rinse, dry thoroughly and proceed with sterilization. **Evacuators:** Add 60ml per 3.8L of water. Insert evacuator hose, turn system on full open for one minute. Suggested maintenance twice weekly.

- Do not use disinfectants together with Enzymax. - Do not use Enzymax in the Thermodesinfector unit. Change solution at least once a day

**DANGER****H318** Causes serious eye damage. **H334** May cause allergy or asthma symptoms or breathing difficulties if inhaled. **P305+P351+P338** IF IN EYES: Rinsecautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P310** Immediately call a POISON CENTER/doctor. **P342+P311** If experiencing respiratory symptoms: Call a POISON CENTER/doctor. Dispose of contents/container in accordance with local/regional/national/international regulations. **Safety Information:** Contains Subtilisin, Amylase, alpha- May produce allergic reaction. **SDS** available on request or at: www.hu-friedy.eu**24H Emergency number:** INTERNATIONAL +49 - (0) 6132 - 84463, GBK GmbH (24h - 7d/w - 365d/a) / In England and Wales: NHS Direct - 0845 4647, In Scotland: NHS 24 - 08454 24 24 24**FR ENZYMAL LIQUID (IMS-1222, IMS-1224, IMS-1226)****ENZYMAL Détergent liquide enzymatique****Volume net:** IMS-1224: flacon pompe de 0,95 L
IMS-1226: flacon de recharge de 3,8 L
IMS-1222: 40 berlingots. de 10 ml (Concentre)

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