**GB ENZYMEX EARTH LIQUID (IMS-1336)****Application:** Enzymatic Cleaner liquid**Net Contents:** IMS-1336: 3,8 liters bottle

**Instructions for use:** Shake well before using. Wear gloves and eye protection. Use with water 32°–60°C, demineralized water is ideal. **Presoak:** Add 30ml of concentrate per 3.8L of water. Bottles with pump insert: pump one time per liter of water. Soak for 3–5 minutes. Rinse, dry thoroughly and proceed with sterilization **Ultrasonic Cleaning:** Add 30ml of concentrate per 3.8L of water. Bottles with pump insert: pump one time per liter of water. After cleaning, rinse, dry thoroughly and proceed with sterilization. **Evacuators:** Add 60ml per 3.8L of water. Insert evacuator hose, turn system on full open for one minute. Suggested maintenance twice weekly.

- Do not use disinfectants together with Enzymax. - Do not use Enzymax in the Thermodesinfector unit. Change solution at least once a day.

**DANGER**

**H334** May cause allergy or asthma symptoms or breathing difficulties if inhaled. **H318** Causes serious eye damage.

**P280** Wear protective gloves/protective clothing/eye protection/face protection. **P260** Do not breathe vapour/spray. **P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P310** Immediately call a POISON CENTER/doctor. **P420** Store away from other materials. **P501** Waste disposal according to local regulations.

SDS available on request or at: [www.hu-friedy.eu](http://www.hu-friedy.eu) **24H Emergency number:** INTERNATIONAL +49 - (0) 6132 - 84463, GBK GmbH (24h - 7d/w - 365d/a) / In England and Wales: NHS Direct - 0845 4647, In Scotland: NHS 24 - 08454 24 24 24

**FR ENZYMEX EARTH LIQUID (IMS-1336)****Application:** ENZYMEX Détergent liquide enzymatique**Volume net:** IMS-1336: flacon de 3,8 L

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