**GB ENZYMAX EARTH POWDER PAX (IMS-1332,IMS-1333 IMS-1332SP)**

**Application:** Enzymatic Cleaner on Powder base  
**Net Contents:** IMS-1332: 32 Powder packets 14 g (water-soluble) - IMS-1333: 96 Powder packets 14 g (water-soluble) - IMS-1332SP: 3 Powder packets 14 g (water-soluble)  
**Instructions for use:** Wear gloves and eye protection. Use with water 32°- 60°C, demineralized water is ideal. **Presoak:** Add 1 packet per 3.8L of water. Soak for 3-5 minutes. **Linen Presoak:** Add 1 to 2 packets per 3.8L. **Ultrasonic Cleaning:** Add 1 packet per 3.8L of water. Rinse thoroughly, dry and proceed with sterilization. **Evacuators:** Add 2 packets per 3.8L of water. Insert evacuator hose, turn system on full open for one minute. Suggested maintenance twice weekly. Do not use disinfectants together with Enzymax. - Do not use Enzymax in the Thermodesinfector unit. Change solution at least once a day.

**DANGER**

**H315** Causes skin irritation. **H318** Causes serious eye damage. **H334** May cause allergy or asthma symptoms or breathing difficulties if inhaled.

**P261** Avoid breathing dust/fume/gas/mist/vapours/spray. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P302+P352** IF ON SKIN: Wash with plenty of water. **P332+P313** If skin irritation occurs: Get medical advice/attention. **P362+P364** Take off contaminated clothing and wash it before reuse. **P304+P340** IF INHALED: Remove person to fresh air and keep comfortable for breathing. **P342+P311** If experiencing respiratory symptoms: Call a POISON CENTER/doctor. **P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P310** Immediately call a POISON CENTER/doctor. **P501** Disposal in accordance with local regulations.

SDS available on request or at: www.hu-friedy.eu  
**24H Emergency number:** INTERNATIONAL +49 - (0) 6132 - 84463, GBK GmbH (24h - 7d/w - 365d/a) / In England and Wales: NHS Direct - 0845 4647, In Scotland: NHS 24 - 08454 24 24 24

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**Numero telefono di emergenza:** +49 - (0) 6132 - 84463, GBK GmbH (24h - 7d/w - 365d/a)
Hu-Friedy Mfg. Co., LLC